Xblitz®

USER'S MANUAL

[ALCONTROL ULTRA]

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User's manual in English language - EN

INTRODUCTION

Thank you for purchasing our product Xblitz ALControl Ultra. We hope it will meet your expectations and ensure safety as well as certain and accurate measurement of the alcohol level in an exhaled air.

01. SCREEN

Test result is displayed in per mil rounded to 3 decimal places.

READY

WARN

LOW BATTERY
LEVEL

1. **READY (green)** – Green diode will turn on when the device is ready for the measure. It will turn on also when the test result will be below permitted level.
2. **LOW BATTERY LEVEL (yellow)** – Yellow diode will turn on when the battery level is too low. Replace the 9V alkaline battery.
3. **WARN (red)** – Red diode will turn on when the test result is above the permitted level (0.20 per mil).

02. DEVICE DESCRIPTION

EN

1. Mouthpiece
2. Screen
3. Battery cover (back of the device)
4. Power on / off button
5. Breathe out (do not cover)

NEVER DRIVE AFTER ALCOHOL CONSUMPTION

03. USAGE

1. Insert the mouthpiece into the adequate slot.
2. Press "power on/off" button located on the right side of the device, under the screen. The screen will turn on and the "on" command will be displayed together with the sonic signal. After this there will be a number of already taken tests shown on the display for a short while. Since that moment the breathalyzer starts counting which takes around 15-35 seconds. It is a time for a sensor to heat up and get prepared for taking a measure.
3. After the sonic signal and "bLo" command displayed on the screen start to blow at the same rate into the mouthpiece until you hear the sonic signal again. At this moment, after the sonic signal and the display command if you will not start to blow within 20 seconds, the device will turn off automatically. In this case turn the device on again.
4. After the rounded line will be shown on a display (3-22 seconds), the test results will be visible (in per mil) for 20 seconds together with the sonic signal. When the test result is more than 0.20 per mil, the "WARN" (red diode) command will be displayed on the screen together with the sonic signal. The test result will be shown for 20 seconds (first 15 seconds adequate diodes will blink, last 5 seconds only the warn command on the display). If you press "power on / off" button, when the green or red diode will not blink, the device will start counting to get prepared for the next measure.
5. The device will show "off" command again and turn off automatically together with sonic signal.
6. For successful measurement follow the point 2-4.
7. Wait around 2 minutes between each measures.

04. WARNING NOTIFICATIONS


0.20	If the test result is above 0.20 per mil, "WARN" command will be displayed, together with sonic signal.
HOT	If the test result is above 4.00 per mil, "HOT" command will be displayed, together with the sonic signal.

CAL	When the time of the measurement exceeds 500 the "CAL" command will be displayed. Even if the device will be still working, it should be re-calibrated. In such case contact your seller or distributor to use the service center.
BAT	Battery level is too low to take a measure. The device will turn off automatically after displaying this command. Replace the battery for the new, 9V alkaline one.
FLO	The user did not exhaled an air long enough, which means that he did not blow stably for 3-5 seconds. Press power on / off button to take next measure.
OUT	If the temperature among the device goes beyond the operation temperature forks (5°-40°), "OUT" command will be displayed and the device will be turned off automatically. Store the device in a place, where the temperature is compatible with the operation temperature.


05. REPLACING THE BATTERY

Push the battery cover (located on the back fo the device) downwards and pull it, so it gets off the cover. Illustrations are on the next page.

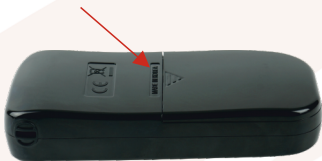NOTE: Make sure that you insert the battery correctly, so the (+) and (-) signs are locaed on the right side.

Pic. 1

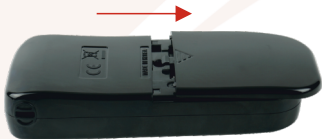Pic. 2

Pic. 3

06. PRECAUTIONS

NOTE: Never blow into the device right after the alcohol consumption. Such action may cause sensor combustion and is not covered by the warranty. Wait at least 20 minutes after eating or drinking before you run the test. Any kind of alcohol remaining in the mouth or saliva may interfere correct measurement and needs 20 minutes to be dispersed. It also refers to the various alcohol preparations such as mouthwash liquid etc.

- 1.** Avoid taking a test outside during strong wind or in the room with polluted air.
- 2.** When there is a "BAT" command displayed on the screen, it is recommended to replace the 9V alkaline battery.
- 3.** Do not blow the cigarette smoke into the device, as well as pieces of food, liquids, as it may cause sensor damage.
- 4.** Do not use the device as the tool to indicate whether you or any other user is able to drive any mechanical vehicle.
- 5.** Store the device within the temperature range: 0°-40°C. The breathalyzer is designed to be used within the temperature range: 5°-40°C. Store the device within those given temperature scopes.
- 6.** Correlation between the alcohol level in an exhaled air and concentration of alcohol in the blood depends on many variables e.g. environmental factors (air quality, wind, humidity, temperature etc.) as well as health condition of the tested person.
- 7.** At normal usage, the device needs renewed calibration once every 6 months (not less than once per 12 months) or after 500 tests. In order to possess a certain results contact your distributor or service center to recalibrate the unit.
- 8.** The measurement results cannot be used as an evidence in court. The driver bears the full responsibility for driving under the influence of alcohol.
- 9.** Do not use this device in combination with ignition blockade in the vehicle.
- 10.** Measurement result may differ dependently on many factors. Possessing a certain result once, does not mean that taking a measure in a short time will result equally.
- 11.** Do not let yourself or anyone else drive after alcohol consumption. Always use a taxi or other vehicle with sober driver when you or anyone else consumed alcohol.

EN

07. TECHNICAL SPECIFICATION

Sensor:	Intelligent electrochemical sensor
Accuracy:	+/- 0.005 per mil
Detection range:	/ 0.00 ~ 4.00‰ per mil
Heat-up time:	15-35 seconds
Reaction time:	3-22 seconds
Dimensions:	125mm x 56mm x 24mm
Weight:	87g (131g with battery)
Power supply:	9V alkaline battery
Calibration:	Once for 6 months (or 500 measures)
Warranty:	5 years

08. WARRANTY / CLAIMS

The product is covered by the 60-months warranty. Warranty conditions can be found on the website: <https://xblitz.pl/warranty>

Complaints should be reported via the complaint form located on the website: <https://reklamacje.kgktrade.pl/>

Details, contact and service address may be found at: www.xblitz.pl

Serial number.....

The warranty is only valid for regular servicing and calibration of the device at an authorized service center. No less than once a year. The device from the novelty is properly calibrated. Each breathalyzer offered on the market requires periodic calibration, ie every 12 months (recommended every 6 months), the device should be configured to correctly interpret the amount of alcohol in the exhaled air. The need for recalibration is not a device disadvantage and is not covered by the warranty. The device not regularly calibrated will show incorrect results.

EN

Importer:

KGK Trade Sp. z o.o. sp.k
os. Teatralne 24
31-946 Cracow
Poland
Made in P.R.C.

RoHS

CALIBRATION

<i>Seller's details:</i>	<i>Calibration date:</i>
<i>Seller's details:</i>	<i>Calibration date:</i>
<i>Seller's details:</i>	<i>Calibration date:</i>
<i>Seller's details:</i>	<i>Calibration date:</i>
<i>Seller's details:</i>	<i>Calibration date:</i>
<i>Seller's details:</i>	<i>Calibration date:</i>
<i>Seller's details:</i>	<i>Calibration date:</i>
<i>Seller's details:</i>	<i>Calibration date:</i>

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