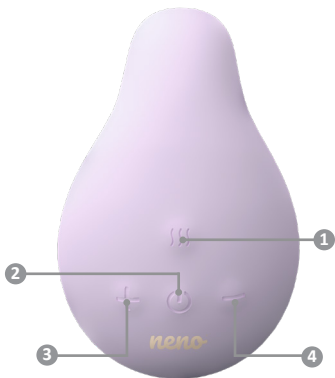
nenò[®]

Pesca

A

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Szanowny Kliencie,

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USER MANUAL

Dear customers,

Thank you for choosing the Neno Pesca. The device you have purchased is a lactation massager with a heating function, which is ideal for the prevention of breast inflammation, brings relief from lactation stasis, reduces breast pain and swelling and also can increase the amount of produced milk. The device should be used in breast and around this area.

01. PRECAUTIONS

Fully charge the massager before using it for the first time. Never use the device while it is charging.

Use only the supplied accessories. Use of any other charger may cause damage to the device, short circuit, electric shock or fire.

Keep away from children. This is not a toy.

Never use the device if it is damaged. Contact an authorised service centre.

Make sure the massager and your hands are dry before connecting to charge, and keep the device away from water while charging.

Never fully submerge the device under water.

02. BUTTON DESCRIPTION

SEE FIGURE A

1. Heating function button
2. Power button
3. Button to increase vibration intensity
4. Button to reduce vibration intensity

03. USE OF THE DEVICE

Switch on the massager by pressing and holding the power button for 2 seconds. To change the type of vibration press the power button selecting from 5 levels in sequence. Reduce or increase the intensity of the vibration using the "+" and "-" buttons.

To turn off the vibration press and hold the power button.

Press and hold the "heating function" button for 2 seconds to switch the heating function on or off. To change the heating intensity press the button again selecting one of the 2 levels.

The device allows you to use both functions independently, so you can use only vibration or heating, or both options at the same time.

To charge your device, connect the power cable to the USB port on your computer, adapter or other power source. Make sure the cable is properly connected to the device. When properly connected for charging, the power button will blink blue to indicate that the device is charging. When the device has finished charging the blue light will stop blinking.

04. SPECIFICATIONS

Material: silicone | **Battery:** Li-ion 500 mA | **Charging time:** approx. 1.5 hours | **Charging:** magnetic charger | **Heating and vibration time simultaneously:** up to 60 minutes | **Dimensions:** 100x65x36 mm | **Weight:** 117 g, 2 heating levels | 5 vibration levels | 3 vibration intensity levels

05. KIT CONTENTS

Lactation massager, Magnetic charger, Velvet pouch, User manual

06. CLEANING AND STORAGE

It is recommended to clean with a soft, water-moistened cloth before and after each use. Make sure the device is switched off and not plugged in to charge. Store the massager in a dry and cool place.

07. WARRANTY CARD

Dear Customer, thank you for purchasing our massager Neno Pesca. If you have any problems with the operation of the device under normal conditions, please contact an authorised Neno brand service centre or distributor. Keep your warranty card in case of repair.

The product comes with a 24-month guarantee. Warranty conditions can be found at: <https://neno.pl/gwarancja>

Details, contact and service address can be found at: <https://neno.pl/kontakt>

Specifications and contents are subject to change without notice. We apologise for any inconvenience.

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EN

The crossed out trash can symbol indicates that unusable electrical or electronic devices, its accessories (such as power supplies, cords) or components (for example batteries, if included) cannot be disposed of alongside with household waste. In order to dispose of the devices or its components (for example, batteries) deliver the device to the collection point, where it will be accepted free of charge. Disposal is subject to the recast version of the WEEE Directive (2012/19/ EU) and the Directive on batteries and accumulators (2006/66 / EC). Proper disposal of the device prevents degradation of the natural environment. Information about the collection points of the facilities is issued by the competent local authorities. Incorrect disposal of waste is subject to penalties provided for by the law in force in the given area.

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